

look inside to  
**SEE WHAT'S  
NEW!**



# READYWRAP™

MANUFACTURED BY SOLARIS

 Distributed in Canada by  
**LYMPHEDEMA  
DEPOT LTD**

*Medical Binder*

[www.LymphedemaDepot.com](http://www.LymphedemaDepot.com)

# READYWRAP

INTRODUCING THE ALL NEW

## All Units

Per your request, Velcro closures are now color-coded to simplify donning!

## Thigh Unit

Now available in standard and custom sizes!

## Knee Unit

Connects the calf to the thigh for full leg coverage! Available in standard and custom sizes to meet all of your clients' needs!

## Calf Unit

NEW design provides greater contouring! Available in standard and custom sizes!

## Foot Unit

NEW simplified design fits into most shoes! NEW left and right options provide better contouring! Standard and custom sizes.



# Table of Contents

- 2 ReadyWrap Full Leg Highlights
- 3 Terms, Pricing, Returns and Warranties
- 3 Contraindications & Materials
- 4 ReadyWrap Products
- 6 ReadyWrap Sizing
- 7 Silver Liners



## Lymphedema Depot

Telephone: 1-905-687-8500

Fax: 1-905-687-8224

Email: [Info@LymphedemaDepot.com](mailto:Info@LymphedemaDepot.com)

## Terms, Pricing, Returns and Warranties

**Terms:** Net 30 days for established dealer accounts; contact us for an Account and Credit Application. All other orders require prepayment by cleared cheque or credit card.

**Pricing:** Please contact us for a current Canadian pricelist. All prices are subject to applicable Canadian sales taxes (Reg. #83343 7221). Prices are subject to change without prior notification

**Returns:** Lymphedema Depot strives to provide the finest quality products and service; we will work with our customers to resolve any issue with our products. Please inspect your order promptly upon receipt. An order discrepancy or any product found defective at time of sale may be returned for exchange within 7 days of receipt with a pre-arranged return authorization. Orders cancelled after shipment will incur round-trip freight charges. Shipping charges are non-refundable. Custom-manufactured garments cannot be cancelled or returned once production has begun.

**Warranties:** ReadyWrap products are warranted against manufacturer defect for six months.

## Contraindications

- Arterial insufficiency or degeneration
- Deep Vein Thrombosis
- Untreated congestive heart failure, localized or systemic infection, or cancer
- Absent or severely impaired sensation

## Materials

ReadyWrap: Nylon, Lycra, Elastane, Polyurethane, Velcro hook

Silver Liners: Nylon, Acrylic, Polyester, Spandex, 





### **A. RW-LE-AB** **ReadyWrap Foot**

The simple two-strap design fits into most shoes and allows for easier donning. Left and right specific designs enhance contouring. ReadyWrap foot is the perfect alternative for clients who have difficulty donning elastic stockings or whose skin may be compromised by shear forces. Like all of our ReadyWraps, the low stretch material provides a low resting / high working pressure similar to industry standard short-stretch bandaging. Standard sizing accommodates ankle girths up to 38cm, custom designing is also available.

### **B. RW-LE-BD** **ReadyWrap Calf**

Maximum therapeutic benefits, excellent contouring, and complete comfort are the unique features that distinguish the ReadyWrap calf unit. The strapping system mimics the 50% overlap of short-stretch bandaging and reduces shear force pressure when being applied to the limb. Velcro placement is designed to minimize strap shifting and enhance ease of donning. Low-stretch materials provide simultaneous low resting / high working pressure maximizing the calf muscle function, especially when exercising or walking. This unit may be worn alone or over the top of compression hosiery.




### **C. RW-LE-DE** **ReadyWrap Knee**

The ReadyWrap Knee unit provides support for your clients with localized patella swelling, or acts as a bridge connecting the calf and thigh units to provide full leg coverage. Please note that when used to provide full leg coverage, the knee unit should be donned last so the client can easily adjust it when in prolonged sit or stand positions. It's simple, 3-strap design, makes this unit easy to don, and the 50% overlap mimics short-stretch bandaging. Available in standard and custom sizes.

### **D. RW-LE-EG** **ReadyWrap Thigh**

The perfect addition to our knee, calf, and foot units for clients with full leg edema. When used by itself, it is also a great option to prevent localized swelling from the superior patella border to the groin. The strap system's 50% overlap mimics short-stretch bandaging. The low-stretch fabric helps ensure low resting and high working pressure. Our experience has shown that for those clients who have a relatively straight leg, the proximal strap is best placed (straight) across. However, for clients with a cone or V-shaped thigh, having the strap create an X-shape prevents slippage.

# ReadyWrap Standard Sizing

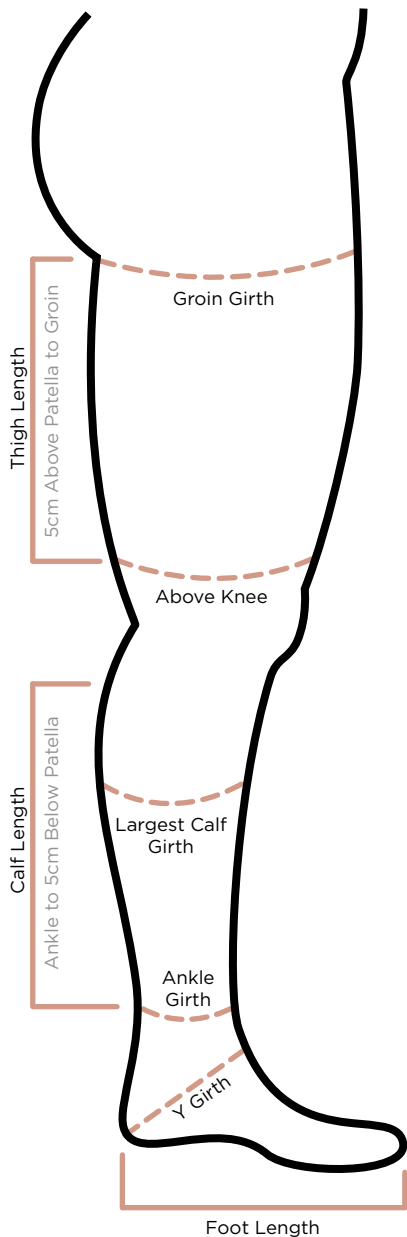

## ReadyWrap Thigh

Description	Length	Above Knee	Groin Girth	Item Code
Reg. Small	25 cm	40 - 55 cm	50 - 70 cm	RW-LE-EG-S25
Tall Small	30 cm	40 - 55 cm	50 - 70 cm	RW-LE-EG-S30
Reg. Med.	25 cm	50 - 65 cm	60 - 80 cm	RW-LE-EG-M25
Tall Med.	30 cm	50 - 65 cm	60 - 80 cm	RW-LE-EG-M30
Reg. Large	25 cm	60 - 75 cm	70 - 90 cm	RW-LE-EG-L25
Tall Large	30 cm	60 - 75 cm	70 - 90 cm	RW-LE-EG-L30
Reg. XL	25 cm	70 - 85 cm	80 - 100 cm	RW-LE-EG-XL25
Tall XL	30 cm	70 - 85 cm	80 - 100 cm	RW-LE-EG-XL30

## ReadyWrap Knee

Description	Calf Girth	Above Knee	Item Code
Small	28 - 38 cm	40 - 55 cm	RW-LE-DE-S
Medium	38 - 43 cm	50 - 65 cm	RW-LE-DE-M
Large	43 - 48 cm	60 - 75 cm	RW-LE-DE-L
Extra Large	48 - 68 cm	70 - 85 cm	RW-LE-DE-XL

## ReadyWrap Calf

Description	Length	Ankle Girth	Calf Girth	Item Code
Reg. Small	30 cm	18 - 23 cm	28 - 38 cm	RW-LE-BD-S30
Tall Small	35 cm	18 - 23 cm	28 - 38 cm	RW-LE-BD-S35
Reg. Med.	30 cm	20 - 28 cm	38 - 43 cm	RW-LE-BD-M30
Tall Med.	35 cm	20 - 28 cm	38 - 43 cm	RW-LE-BD-M35
Reg. Large	30 cm	25 - 33 cm	43 - 48 cm	RW-LE-BD-L30
Tall Large	35 cm	25 - 33 cm	43 - 48 cm	RW-LE-BD-L35
Reg. XL	30 cm	30 - 38 cm	48 - 68 cm	RW-LE-BD-XL30
Tall XL	35 cm	30 - 38 cm	48 - 68 cm	RW-LE-BD-XL35

## ReadyWrap Foot

Desc.	Foot	Length	Y Girth	Ankle Girth	Item Code
Small	L	21 cm	28 - 33 cm	18 - 23 cm	RW-LE-AB-L-S
Small	R	21 cm	28 - 33 cm	18 - 23 cm	RW-LE-AB-R-S
Medium	L	23 cm	30 - 38 cm	20 - 28 cm	RW-LE-AB-L-M
Medium	R	23 cm	30 - 38 cm	20 - 28 cm	RW-LE-AB-R-M
Large	L	26 cm	35 - 43 cm	25 - 33 cm	RW-LE-AB-L-L
Large	R	26 cm	35 - 43 cm	25 - 33 cm	RW-LE-AB-R-L
XL	L	29 cm	40 - 48 cm	30 - 38 cm	RW-LE-AB-L-XL
XL	R	29 cm	40 - 48 cm	30 - 38 cm	RW-LE-AB-R-XL

# ReadyWrap Custom Sizing

We know that while most of your clients will fit into our standard sizes, you'll occasionally have one who would benefit more from a custom garment. That's why we're proud to offer custom sizing for all of our ReadyWrap Foot, Calf, Knee, and Thigh units. If you happen to run into a particularly difficult case, Solaris Design Team is always happy to discuss solutions with you, too. Just give us a call at (905) 687-8500 if you have any questions. Custom forms are available in our price lists and order forms catalog. Give us a call or send us an email if you need a current copy.



**A. LN-LE-AD**  
**Solaris Silver Liner Below Knee**


**B. LN-LE-AG**  
**Solaris Silver Liner Thigh-High**

Solaris is proud to offer a liner specifically designed to be used under compression garments. Every Solaris Silver Liner is manufactured with unique silver, antimicrobial properties for maximum comfort and protection. Solaris Silver Liners are manufactured in the USA and are available in below knee and thigh-high versions. Both versions accommodate girths up to 75cm.

# READYWRAP

MANUFACTURED BY SOLARIS

Visit us at [www.LymphedemaDepot.com](http://www.LymphedemaDepot.com)

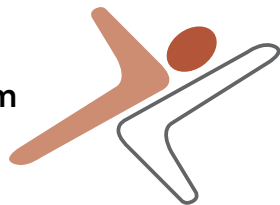

Follow us on:



[facebook.com/lymphedemadepot](https://facebook.com/lymphedemadepot)



[twitter.com/lymphedemadepot](https://twitter.com/lymphedemadepot)

Lymphedema Depot Ltd.  
Mail to: 12-111 Fourth Avenue, Suite 347  
St. Catharines, ON L2S 3P5

**Change Service Requested**