

## Measuring Guidelines for Facial Units

### Positioning Your Client When Measuring for Facial Garments

Tribute garments are most commonly worn while the client is sleeping, however, clients with facial swelling may be measured in a seated posture.

#### Measuring:

*Please record the needed measurements in the designated areas on the Facial Order Form.*

- A:** Measure the length from one ear tip to the other.
- B:** Measure the head circumference.
- C:** Measure the distance from the outside crease of the right eye to the outside crease of the left eye.
- D:** Measure the length of the nose.
- E:** Measure the length at the midline from the nose to the upper lip.
- F:** Measure the length at the midline from the lower lip to the chin.
- G:** Measure the length from the sterno-clavicular notch to the chin.
- H:** Measure from the right ear lobe to the right nostril.
- I:** Measure from the left ear lobe to the left nostril.
- J:** Measure the distance from the right TMJ to the chin.
- K:** Measure the distance from the left TMJ to the chin.
- L:** Measure the length from the right ear lobe to the HPS (High Point of Shoulder).
- M:** Measure the length from the left ear lobe to the HPS.
- N:** Measure the neck circumference.

## Measuring Guidelines for Hand Units

### Positioning Your Client When Measuring for Hand Garments

The Hand Order Forms are Right / Left specific.

Position your client's hand on a copy of the order form. Align the wrist crease to approximate the Center Wrist Line on the copy of the order form. Trace around the client's hand with a pen or pencil, keeping the tip of the pen/pencil close to the digits and hand. Have the client remove their hand from the copy of the Hand Order Form.

#### Measuring Circumferences:

*Please record the needed measurements in the designated areas on the specific Hand Order Form.*

- A:** Measure the wrist circumference over the styloid.
- B:** Measure the palm circumference across the metacarpal heads.
- C:** Measure the proximal circumference of the thumb.
- D:** Measure the proximal circumference of the index finger.
- E:** Measure the proximal circumference of the middle finger.
- F:** Measure the proximal circumference of the ring finger.
- G:** Measure the proximal circumference of the small finger.

**Solaris Garments, including the Tribute™, are distributed in Canada exclusively by:**



**Customer Service:** (905) 687 – 8500  
**Fax :** (905) 687 – 8224  
**Email:** [Info@LymphedemaDepot.com](mailto:Info@LymphedemaDepot.com)  
**Web:** [www.LymphedemaDepot.com](http://www.LymphedemaDepot.com)