


# PROVIDING A HAND TRACING - TIPS & GUIDELINES

For ordering a Solaris Tribute Glove, or Tribute Arm with Custom Digit Spacers

**2** With your wrist crease positioned along the baseline you drew, lay your hand on the page in as relaxed a position as you can, making sure that the palm is pressed flat to the page. Try to avoid stiffening your fingers and spreading them out on the page, as we are aiming for the closest approximation we can get to your hand posture while you are relaxed and/or sleeping.

**1** On a blank sheet of paper, near the bottom edge, mark a "baseline" that you will place the crease of your wrist on to take your hand tracing. Mark a scale on the page to indicate 1 cm in case your tracing loses its scale during faxing or emailing.

**3** When you trace, be sure to keep your pen or pencil in a straight up and down position so that the outline you make is around the hand, and not under it. Don't tip the pen inward so that the outline is too tight around the hand.

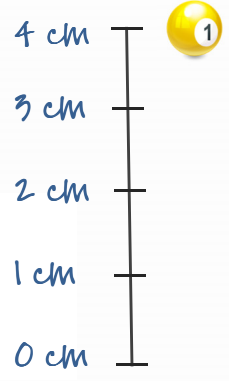


**5** Measure circumference around the base of the thumb and all fingers.

**6** Measure circumference around the palm at the level of the thumb web with the hand in a neutral position.

**7** Measure circumference around the wrist.

**ALL DONE!**

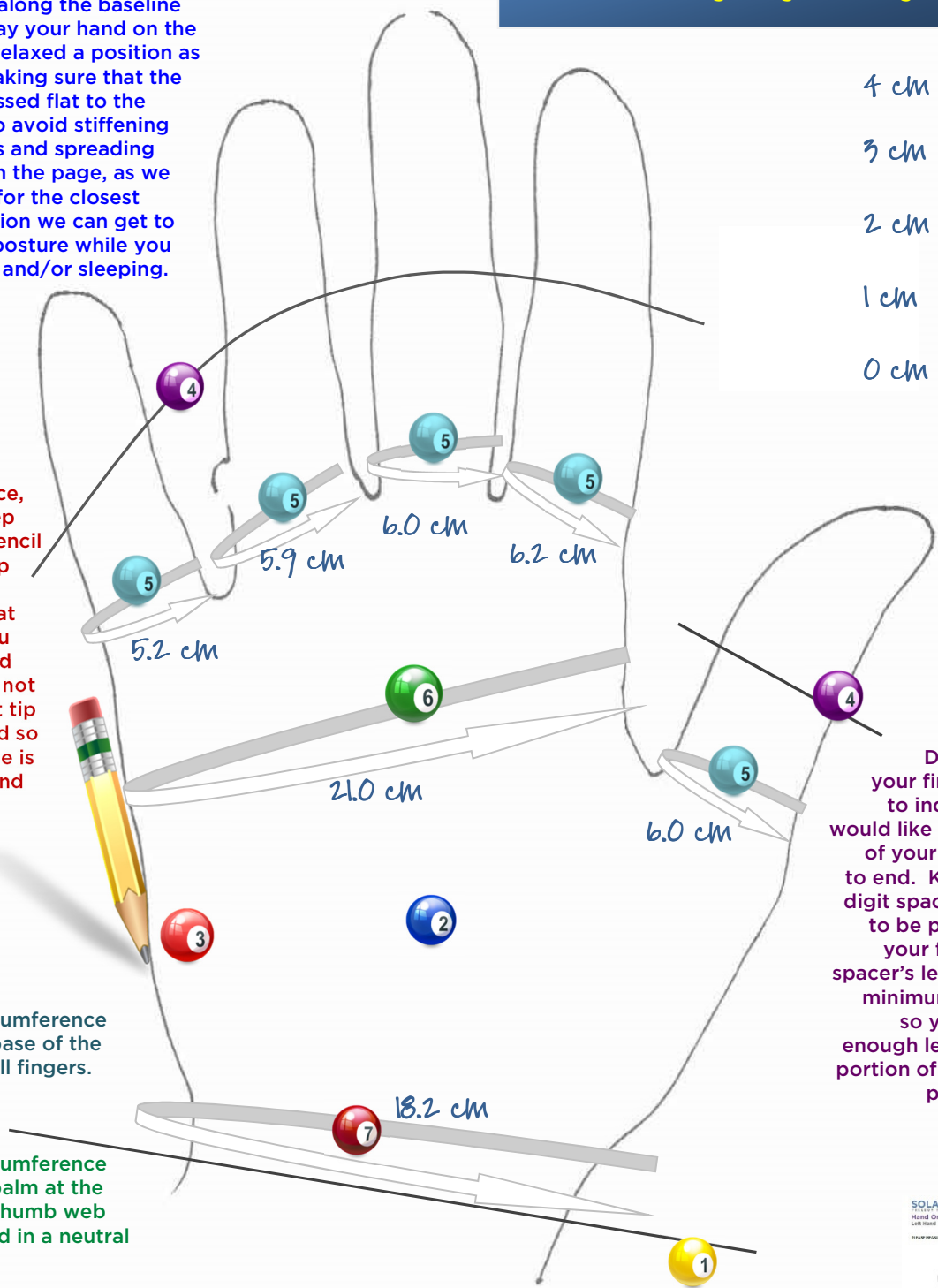

Draw a line across your fingers and thumb to indicate where you would like the hand portion of your Tribute garment to end. Keep in mind that digit spacers are intended to be placed BETWEEN your fingers, and each spacer's length should be a minimum of about 2 cm, so you have to allow enough length in the hand portion of your garment to place the spacers.

Note: this is an alternate method of providing details for gloves and digit spacers. You can also use the **HAND MEASURING FORM** →