

SWELL SPOTS™

MANUFACTURED BY SOLARIS



> user guide

YOUR SWELL SPOT

Swell Spots are professionally designed treatment pads that tuck into garments or under bandaging to address particularly troublesome areas of edema, fill contours, or diffuse pressure points and break up fibrosis. Please consult with your medical professional for individual usage.



CARE INSTRUCTIONS & HELPFUL HINTS

- › Use the grey fabric against your skin.
- › Wash weekly or more frequently, as needed.
- › Machine wash with mild detergent on perm press.
- › Double rinse to remove any soap residue.
- › Never use additives, such as bleach or fabric softener.
- › Tumble dry on perm press to re-fluff the foam.
- › Place clean tennis balls in dryer to fluff up the foam.
- › Stop use if rash, numbness, skin color changes are noticed.



MATERIALS



- › 63% Nylon, 23% Polyester, 14% Lycra
- › Latex-free
- › Medical grade Polyurethane foam
- › Made in the USA

RETURNS & WARRANTY

Within two weeks of the purchase date Solaris will replace defective products. Please contact Solaris directly. Solaris does not accept returns otherwise. In an effort to keep prices low, Solaris does not warrant Swell Spots products.

INDICATIONS

- › Lymphedema
- › Hypertrophic Scarring
- › Acute / Chronic Edema
- › Indurate Tissue

CONTRAINDICATIONS

- › Arterial Insufficiency or Degeneration
- › Deep Vein Thrombosis (DVT)
- › Untreated Congestive Heart Failure or Cancer
- › Untreated Localized or Systemic Infection
- › Absent, or Severely Impaired Sensation



F R E E D O M T O L I V E

6737 W. Washington St, Ste 3260
West Allis WI 53214 USA

T: (855) 892-4140 | F: (414) 892-4150 | E: info@solarismed.com
www.solarismed.com

Distributed in Canada exclusively by:

**LYMPHEDEMA
DEPOT LTD** 
Chronic Edema & Wound Care Products

www.LymphedemaDepot.com
1-855-857-8500

REV 10/11